



Certificate of Quality Control

ABACard® P30

Specifications:

Each lot of ABACard® P30 must be tested. QC will consist of testing a reagent blank and a known semen standard swab (1:100 dilution). The 1:100 dilution of semen standard must be positive and the reagent blank must be negative.

Directions:

Using deionized water and at least 500µl of a liquid semen sample, create a 1:100 dilution of semen. Dip one swab into each dilution and a second swab into the water only. Allow the swabs to dry thoroughly (e.g., overnight). Place ¼ of each swab into separate tubes of 650µl buffer solution provided in the kit. Follow the instructions for testing samples as prescribed in the SOP.

Lot # _____ Expiration Date _____

QC analyst _____ Date QC completed _____

Required records:

- Certificate of Quality Control
- Photo of P30 card results

Results

SAMPLE	POSITIVE	NEGATIVE
1:100		
Reagent blank		

Results of Testing: Pass Re-test (Choose One)

Results of Re-testing: Pass Fail Not Needed (Choose One)

Comments:

Approval of COQC Record
by Chief Criminalist, DNA
Team Leader, or designee: _____

Date* _____

**If passed, this is the date released*